

Manufactured for Immuno-Biological Laboratories Inc.
(IBL-America)
8201 Central Avenue, NE, Suite P
Minneapolis, MN 55432
Tel: 763-780-2955
Toll Free: 1-888-523-1246



Current version No.:	Updates V 147.3
Previous version:	V 147.2
Update in section:	General Updates

Leishmania IgG ELISA

CONTENTS

- 1 INTENDED USE
- 2 BACKGROUND
- 3 TEST PRINCIPLE
- 4 KIT COMPONENTS
- 5 MATERIAL REQUIRED BUT NOT SUPPLIED
- 6 STORAGE AND STABILITY
- 7 TEST PROCEDURE
 - 7.1 Evidence of Deterioration
 - 7.2 Sample Preparation and Storage
 - 7.3 Preparation of Kit Reagents
 - 7.4 Overview - Test Procedure
 - 7.5 Manual Test Procedure
 - 7.6 Automated Test Procedure
 - 7.7 Positive Control / Accuracy Control
- 8 TEST EVALUATION
 - 8.1 Criteria of Validity
 - 8.2 Cut-off Calculation
 - 8.3 Borderline Ranges
- 9 PERFORMANCE CHARACTERISTICS
 - 9.1 Sensitivity and Specificity
 - 9.2 Reproducibility
- 10 SAFETY MEASURES
 - 10.1 Statements of Warning
 - 10.2 Disposal
- 11 REFERENCES

**For Research Use
Only – Not for Use
in Clinical
Procedures**

IBL-America Leishmania IgG ELISA

Enzyme-immunoassay for determination of human antibodies

Leishmania IgG ELISA

Order no.: IB05109

For Research Use Only – Not for Use in Clinical Procedures

1 INTENDED USE

The IBL-America Leishmania IgG ELISA test is a qualitative immunoassay for the demonstration of IgG antibodies in serum or plasma directed against *Leishmania*. The assay is recommended for the sensitive detection of such antibodies in various kinds of samples.

2 BACKGROUND

Leishmaniasis is an infectious disease, caused by protozoic parasites of the genus *Leishmania*. The disease occurs predominantly in tropical and subtropical climate zones. Primarily affected are farm and domestic animals, however the disease can also be transmitted to humans. Leishmaniasis is distributed worldwide with high incidences in Eastern Africa, Southern America and Asia. In Europe, the Mediterranean area is particularly affected. Annually, some 1.5 million cases of cutaneous and another 0.5 million cases of visceral leishmaniases are reported. Around 12 million people are infected with *Leishmania* worldwide.

Dogs and rodents serve as the main reservoirs for *Leishmania*, but also cats, horses, sheep and cattle may be afflicted. Sandflies (*phlebotominae*) or other moth flies (*psychodidae*) transmit the parasites to humans. The incubation period is very variable, ranging from a few weeks up to several years.

ELISA test systems are especially suited for the differential analysis of immunoglobulin classes directed against the virus.

3 TEST PRINCIPLE

The ELISA (Enzyme Linked Immunosorbent Assay) is an immunoassay, which is particularly suited to the determination of antibodies in various kinds of samples. The reaction is based on the specific interaction of antibodies with their corresponding antigen. The test strips of the microtiter plate are coated with specific antigens of the pathogen of interest. If antibodies in the sample are present, they bind to the fixed antigen. A secondary antibody, which has been conjugated with the enzyme alkaline phosphatase, detects and binds to the immune complex. The colourless substrate p-nitrophenylphosphate is then converted into the coloured product p-nitrophenol. The signal intensity of this reaction product is proportional to the concentration of the analyte in the sample and is measured photometrically.

4 KIT COMPONENTS

Test Components	Pieces / Volume
Break apart microtiter test strips each with eight antigen coated single wells, (altogether 96) [MTP] , 1 frame. The coating material is inactivated.	12 pieces
Standard serum (ready-to-use) [STD] , Human serum in protein containing phosphate buffer; negative for anti-HIV Ab, HBs-Ag (Hepatitis B-Virus surface antigen) and anti-HCV Ab; preservative: < 0.1 % sodium azide; colouring: Amaranth O	2 x 2 ml
Negative control serum (ready-to-use) [NEG] , Human serum in protein containing phosphate buffer; negative for anti-HIV Ab, HBs-Ag (Hepatitis B-Virus surface antigen) and anti-HCV Ab; preservative: < 0.1 % sodium azide; colouring: Lissamin Green V	2 ml
Anti-human IgA, IgG or IgM conjugate (ready-to-use) [APC] , Anti-human IgA, IgG or IgM polyclonal antibody, conjugated to alkaline phosphatase, stabilised with protein stabilisation solution; preservative: < 0.1 % methylisothiazolone, < 0.1 % bromnitrodioxane	13 ml
Washing solution concentrate (sufficient for 1000 ml) [WASH] , Sodium chloride solution with Tween 20 and 30 mM Tris/HCl, pH 7.4; preservative: < 0.1 % sodium azide	33.3 ml
Dilution buffer (ready-to-use) [DILB] , Protein containing phosphate buffer with Tween 20; preservative: < 0.1 % sodium azide; colouring: 0.01 g/l Bromphenol blue	2 x 50 ml
Stopping solution (ready-to-use) [STOP] , < 0.1 N sodium hydroxide, 40 mM EDTA	15 ml
Substrate (ready-to-use) [pNPP] , Para-nitrophenylphosphate in solvent free buffer; preservative: < 0.1 % sodium azide	13 ml
Quality control certificate [INFO]	1 page

5 MATERIAL REQUIRED BUT NOT SUPPLIED

- Common laboratory equipment
- For the IgM detection: Rf-Absorbent, order no. IB05998 (20 ml)
- Photometer for microtiter plates with filter, wavelength 405 nm, recommended reference wavelength 620 nm - 690 nm (e.g. 650 nm)
- Microtiter plate washer
- Incubator 37 °C
- Moist chamber
- Distilled water

6 STORAGE AND STABILITY

Reagent	Storage	Stability
Microtiter strips (coated with antigen)	unopened after opening at 2 – 8 °C in closed aluminum bag with desiccant	see expiry date 6 months
Control sera / Standard sera	unopened after opening at 2 – 8 °C	see expiry date 6 months
Conjugate	unopened after opening at 2 – 8 °C	see expiry date 6 months
Dilution buffer	unopened after opening at 2 – 8 °C	see expiry date 6 months
Washing solution	unopened / after opening at 2 – 8 °C working dilution at 2 – 8 °C working dilution at room temperature	see expiry date 2 weeks 1 week
Substrate	unopened after opening at 2 – 8 °C	see expiry date 6 months
Stopping solution	unopened after opening at 2 – 8 °C	see expiry date 6 months

7 TEST PROCEDURE

7.1 Evidence of Deterioration

Optimum results can only be achieved if the instructions are strictly followed. The components of the kit must not be exchanged for reagents of other manufacturers. Standard and control sera are defined exclusively for the test kit to be used and must not be used in other lots. Dilution buffer, washing solution, substrate and stop solution can be used for all IBL-America immunoassays coded IB05xxx irrespective of the lot and the test.

Each IBL-America ELISA coded IB050xxx contains a ready-to-use sample dilution buffer. In some cases the use of special dilution buffers is necessary to guarantee consistent quality and reliable results. The dilution buffers can be used irrespective of the lots.

There are three different conjugate concentrations for each immunoglobulin class (IgA, IgG, IgM), indicated on the label as + (low), ++ (medium) and +++ (high). Conjugates with the same concentration and of the same immunoglobulin class are interchangeable and can be used for other IBL-America ELISAs coded IB05xxx irrespective of the lot and the test. Dilution or alteration of the reagents may result in a loss of sensitivity. Use aseptic techniques when removing aliquots from the reagent tubes to avoid contamination.

Reproducibility of test results is dependent on thorough mixing of the reagents. Agitate the flasks containing control sera before use and also all samples after dilution (e.g. by using a vortex mixer).

Be sure to pipette carefully and comply with the given incubation times and temperatures. Significant time differences between pipetting the first and last well of the microtiter plate when dispensing samples and control sera, conjugate or substrate can result in different pre-incubation times, which may influence the precision and reproducibility of the results. Avoid exposure of reagents to strong light during storage and incubation.

Optimum results can only be achieved if the instructions are strictly followed.

The results of this ELISA are only valid if the lot-specific validation criteria on the quality control certificate are fulfilled.

Adequate washing avoids test unspecificities. Therefore, the washing procedure should be carried out carefully. All of the flat bottom wells should be filled with equal volumes of washing buffer. At the end of the procedure ensure that the wells are free of all washing buffer in order to avoid uncontrolled dilution effects. Avoid foaming!

Reagents must be tightly closed after use to avoid evaporation and contamination. Take care not to mix-up the caps of the bottles and/or vials.

The ELISA test kits are only valid if the lot-specific validation criteria on the quality control certificate are fulfilled.

7.2 Sample Preparation and Storage

Lipaemic, hemolytic or icteric samples (serum or plasma) should only be tested with caution. Obviously contaminated samples should not be tested. Serum or plasma (EDTA, citrate, heparin) collected according to standard laboratory methods are suitable samples. Samples must not be thermally inactivated.

7.2.1 Dilution of Samples

Before running the test, all samples (V_1) must be diluted in dilution buffer (V_2) as follows:

$V_1 + V_2 = 1+100$	add	10 μ l	sample
	to each	1000 μ l	dilution buffer

After dilution and before pipetting into the microtiter plate the samples must be mixed thoroughly to prepare a homogenous solution.

7.2.2 Sample Storage

The samples should not be stored for more than 7 days at 2 – 8 °C. Extended storage is possible at ≤ -20 °C. Avoid repeated freezing and thawing of samples. Diluted samples can be stored at 2 – 8 °C for one week.

7.3 Preparation of Kit Reagents

Bring all reagents to room temperature before testing.

7.3.1 Microtiter Test Strips

The microtiter test strips labeled with abbreviations for pathogen and immunoglobulin class are packed with a dessicant in an aluminum bag. To open the aluminum bag of the microtiter plate please cut off the top of the marked side only, in order to guarantee proper resealing. Take unrequired cavities out of the frame and put them back into the aluminum bag. Close bag carefully to ensure airtight conditions. Do not use the strips if the aluminum bag is damaged or if the bag with remaining strips and dessicant was not properly resealed.

7.3.2 Control Sera / Standard Sera (ready-to-use)

Negative control and standard sera are ready-to-use and must not be diluted any further. For each test run - independent of the number of microtiter test strips to be used – negative control and standard sera must be included. Standard sera should be set up in duplicate. Do not treat negative control and standard sera with Rf-absorbent.

7.3.3 Anti-human IgG AP-Conjugate (ready-to-use)

Conjugates with the same concentration and of the same immunoglobulin class are interchangeable. The required conjugate concentration (+, ++, +++) is indicated on the quality control certificate. Avoid contamination.

7.3.4 Washing Solution (Concentrate)

Dilute washing buffer concentrate (V_1) 1:30 with aqua dest. to a final volume of V_2 . Bottles used for the working dilution should be cleaned regularly. Discard cloudy solutions.

Example:

Buffer concentrate (V_1)	Final volume (V_2)
33.3 ml	1000 ml
1.0 ml	30 ml

7.3.5 Dilution Buffer for Samples (ready-to-use)

Discard cloudy solutions

7.3.6 Substrate (ready-to-use)

Substrate in unopened bottle may have a slightly yellow colouring, which does not reduce the quality of the product. Avoid contamination.

7.3.7 Stopping Solution (ready-to-use)

7.4 Overview - Test Procedure

IBL-America Leishmania IgG

sample dilution¹
1+100

Pipette diluted samples and ready-to-use control /
standard sera into the microtest wells (100 µl)



INCUBATION 60 min./ 37 °C
moist chamber



WASH (4 x 300 µl [DIL] [WASH])²



Pipette conjugate solution [APC] (100 µl)



INCUBATION 30 min./ 37 °C
moist chamber



WASH (4 x 300 µl [DIL] [WASH])²



Pipette substrate solution [pNPP] (100 µl)



INCUBATION 30 min./ 37 °C
moist chamber



Pipette stopping solution [STOP] (100 µl)



READ EXTINCTION at 405 nm

¹Special dilution buffers for the following IBL-America tests:
Borrelia burgdorferi IgG, IgM, EBV EA IgG

²For manual use:
tap plate at the end of the wash procedure on paper towel.

7.5 Manual Test Procedure

1. Place the required number of **cavities in the frame** and prepare a protocol sheet.
2. Add each **100 µl of diluted sample or ready-to-use controls** into the appropriate wells of microtiter test strips. Spare one well for substrate blank, e.g.:

Well	IgG ELISA
A1	substrate blank
B1	negative control
C1	standard serum
D1	standard serum
E1	sample 1 ...
F1	sample 2 ...

3. **Sample incubation** for 60 minutes (+/- 5 min.) at 37 °C (+/- 1°C) in moist chamber
4. After incubation **wash** all wells with washing solution (by automated washer or manually):
 - aspirate or shake out the incubation solution
 - fill each well with 300 µl washing solution
 - aspirate or shake out the washing buffer
 - repeat the washing procedure 3 times (altogether 4 times!)
 - dry by tapping the microtiter plate on a paper towel
5. **Addition of conjugate**
Add 100 µl of the ready-to-use IgG conjugate to the appropriate wells (except substrate blank)
6. **Conjugate incubation** for 30 minutes (+/- 1 min.) at 37 °C (+/- 1 °C) in moist chamber.
7. After incubation **wash** all wells with washing solution (see above).
8. **Addition of substrate**
Add 100 µl of ready-to-use substrate solution to each well (including well for substrate blank!)
9. **Substrate incubation** for 30 minutes (+/- 1 min.) at 37 °C (+/- 1 °C) in moist chamber.
10. **Stopping of the reaction**
Add 100 µl stopping solution to each well, shake microtiter plate gently to mix.
11. **Read extinction**
Read optical density (OD) within 60 minutes at 405 nm against substrate blank, reference wave length between 620 nm and 690 nm (e.g. 650 nm).

7.6 Automated Test Procedure

These ELISA tests also are validated for use with Immunomat (using the following consumables: VT124, VT111, VT112) and suited for processing on similar analyzers. For processing on the Immunomat the current software version including reagent check has to be used. The automated processing is performed analogous to manual use. Please note, that under special working-conditions internal laboratory adaptations of the substrate incubation times may be necessary.

7.7 Positive Control / Accuracy Control

For the periodic verification of the test method, in order to fulfil the requirements of laboratory internal quality management systems, we recommend using IBL-America ELISA controls (cat.-no. IB05xxxCON, see also chapter 5) to determine precision and accuracy of the test runs. The use of IBL-America ELISA controls is described in specific instruction manuals.

8 TEST EVALUATION

For qualitative interpretation of serum samples a lot specific correction factor as well as a lot specific grey zone is calculated by manufacturer for each kit lot. These values can be found on the lot specific quality certificate included in each test kit.

For test run control a standard serum is used in each individual test run. For this control serum a reference value with a validity range is determined by the quality control of the manufacturer. Within this range a correct cut-off interpretation is ensured.

8.1 Criteria of Validity

The substrate blank must be < 0.25 OD

The negative control must produce a negative test result.

The mean OD-value (after subtraction of the substrate blank!) of the standard serum must be within the validity range, which is given on the lot specific qualitycontrol certificate.

The variation of OD-values of the standard serum may not be higher than 20%.

If these criteria are not met, the test is not valid and must be repeated.

8.2 Cut-off Calculation

A lot specific quality control certificate is included in the test kit so that the obtained OD values can be interpreted qualitatively. The substrate blank must be subtracted from all OD values prior to evaluation.

To fix the cut-off ranges multiply the mean value of the measured standard OD with the lot specific correction factor from the quality certificate. Then add and subtract the lot specific grey zone percentage mentioned on the quality certificate to obtain the upper and lower cut-off. The following numbers are an example only, the valid data you will find in the lot-specific QC certificate which comes with each kit.

Lot specific correction factor: 0,805

Lot specific grey zone: 15%

If the measured mean absorbance value of the standard serum is 0.84 OD, the range of the cut-off is:

Lower cut-off: $(0,84 * 0,805) - 15\% = \text{OD } 0,575$

Upper cut-off: $(0,84 * 0,805) + 15\% = \text{OD } 0,778$

8.3 Borderline Ranges

The borderline range indicates the range for borderline test results. Values obtained, when testing a sample, which fall below this range indicate a negative test result; values above the borderline range are interpreted positive. In cases where the results are within the borderline range a definitive interpretation of the result is not possible. In such cases, the test should be repeated in parallel with a follow-up sample taken one to two weeks later (serum pair).

9 PERFORMANCE CHARACTERISTICS

9.1 Sensitivity and Specificity

IBL-America Leishmania IgG

The IBL-America Leishmania IgG ELISA was verified in an internal study. The sensitivity and specificity in both cases exceeded 99%.

9.2 Reproducibility

Leishmania IgG ELISA

Intraassay reproducibility was determined by testing sera of different reactivities 20 times in one test run. Interassay reproducibility was determined by testing sera of different reactivities in 10 independent test runs on 5 different days.

$$\text{Coefficient of Variation (CV \%)} = \frac{\text{Standard deviation}}{\text{Mean value}} \times 100$$

Sample	Mean Value (OD)	Intraassay CV (%)	Mean Value (OD)	Interassay CV (%)
Sample 1	0.368	3.1	0.398	4.9
Sample 2	0.435	2.5	0.460	6.2
Sample 3	1.511	1.5	1.584	4.1

9.3 Cross-reactivities

Leishmania IgG:

To determine detection of cross-reactive antibodies directed against different parameters sera were analyzed with the Leishmania IgG ELISA and a commercially available anti-Leishmania IgG ELISA. Positive sera (9 or 10 sera each) for Epstein-Barr Virus VCA IgG, Cytomegalovirus IgG, Varicella Zoster Virus IgG and Toxoplasma gondii IgG have been tested as well as sera positive for rheumatoid factor (RF) and anti-nuclear antibodies (ANA). Within this internal evaluation no potential cross-reactivities have been observed. Other cross-reactivities cannot be ruled out in general.

9.4 Interfering substances

Leishmania IgG:

To determine detection of interfering substances, sera with different reactivities were analyzed with the Leishmania IgG ELISA. No interferences have been detected for sera with concentrations up to 2.00 g/L hemoglobin, 11.50 g/L lipemia/triglyceride oder 0.201 g/L bilirubin (conjugated and unconjugated).

10 SAFETY MEASURES

10.1 Statements of Warning

The IBL-America test kits are designed for use by qualified personnel who are familiar with good laboratory practice. All kit reagents and specimens should be handled carefully, using established good laboratory practice.

- This kit contains human blood components. Although all control- and cut-off sera have been tested and found negative for anti-HIV-ab, HBs-Ag (*Hepatitis B-Virus-surface Antigen*) and anti-HCV-ab, they should be considered potentially infectious.
- Do not pipette by mouth.
- Do not smoke, eat or drink in areas in which specimens or kit reagents are handled.
- Wear disposable gloves, laboratory coat and safety glasses while handling kit reagents or specimens. Wash hands thoroughly afterwards.
- Sample material and other potentially infectious material should be decontaminated after the test run.
- Reagents should be stored safely and be inaccessible to unauthorized access e.g. children.

10.2 Disposal

Please observe the relevant statutory requirements!

11 REFERENCES

- [1] Goto H. and Lauletta Lindoso JA. (2010) Current diagnosis and treatment of cutaneous and mucocutaneous leishmaniasis. *Expert Rev. Anti Infect. Ther.* 8(4), 419–433 (2010)
- [2] Liu D. and Uzonna JE. (2012) The early interaction of *Leishmania* with macrophages and dendritic cells and its influence on the host immune response. *Frontiers in Cellular and Infection Microbiology*, June 2012, Volume 2, Article 83
- [3] Maalej IA., Chenik M., Louzir H., Ben Aalah A., Bahloul C., Amri F., and Dellagi K. (2003) Comparative Evaluation of ELISAs based on ten recombinant or purified *Leishmania* Antigens for the Serodiagnosis of Mediterranean Visceral Leishmaniasis. *Am. J. Trop. Med. Hyg.*, 68(3), 2003, pp. 312–320
- [4] Roberts LJ., Handman E., and Foote SJ (2000) Leishmaniasis. *BMJ* Volume 321
- [5] Rodríguez-Cortés A., Ojeda A., Francino O., López-Fuertes L., Timón M., and Alberola J. (2010) *Leishmania* Infection: Laboratory Diagnosing in the Absence of a “Gold Standard”. *Am. J. Trop. Med. Hyg.*, 82(2), 2010, pp. 251–256
- [6] Ejazi, S.A., Ali N. (2013) Developments in diagnosis and treatment of visceral leishmaniasis during the last decade and future prospects. *Expert review of anti-infective therapy*, Jan 11 (1), 2013, pp. 79-98, doi: 10.1586/eri.12.148

Manufactured for Immuno-Biological Laboratories Inc.
(IBL-America)
8201 Central Avenue, NE, Suite P
Minneapolis, MN 55432
Tel: 763-780-2955
Toll Free: 1-888-523-1246

