

Product information

Information about other products is available at: www.ibl-america.com



User's Manual

Echinococcus granulosus IgG ELISA

Enzyme immunoassay for the detection and quantitative determination of human IgG antibodies against Echinococcus in serum and plasma.



For in-vitro diagnostic use only



IB79221



96 wells

Contents

1. INTENDED USE3
2. GENERAL INFORMATION.....3
3. PRINCIPLE OF THE TEST.....3
4. LIMITATIONS, PRECAUTIONS AND GENERAL COMMENTS4
5. REAGENTS PROVIDED.....4
6. MATERIALS REQUIRED BUT NOT PROVIDED5
7. SAMPLE COLLECTION AND HANDLING.....6
8. ASSAY PROCEDURE6
9. EVALUATION.....7
10. ASSAY CHARACTERISTICS.....7
11. REFERENCES8
12. ORDERING INFORMATION8
SYMBOLS USED WITH IBL-AMERICA ASSAYS9

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
8201 Central Ave NE, Suite P, Minneapolis, MN 55432
Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com

1. INTENDED USE

The IBL-America Echinococcus IgG Antibody ELISA has been designed for the detection and the quantitative determination of specific IgG antibodies against Echinococcus in serum and plasma. For *in-vitro* diagnostic use only.

2. GENERAL INFORMATION

Echinococcus spp. are 1.4 - 6 mm long tapeworms with 3 - 5 segments. The two species *E. granulosus* and *E. multilocularis* are principally responsible for human disease. These species are the aetiological agents of hydatid disease, a zoonotic infection of worldwide distribution, in man and other intermediate hosts. The definitive host for *E. granulosus* is the dog, with sheep or cattle acting as intermediate hosts following ingestion of contaminated dog faeces. Man is also infected coincidentally by contact with contaminated dog faeces. The definitive hosts for this organism are foxes and dogs, with small rodents acting as intermediate hosts. Infection is mainly acquired by ingestion of contaminated fruits and vegetables. Most of the symptoms of echinococcosis arise from the compression of adjacent host structures by the gradually enlarging hydatid cyst. The preferred form of treatment is surgical removal of the intact cyst. In *E. multilocularis* infections the germinal membrane is not confined within a single cyst, and scolices develop in an uncontrolled manner, often with extensive local invasion and infiltration, particularly in the liver. Unlike classical hydatid disease, this renders little possibility of effective surgical resection. The disease can stay without symptoms over a period of some years. Depending on the extension and the localisation of the cysts, there can be different symptoms like epigastric pain, ikterus, bilious vomiting, thoracodynia or paresis. The lethality caused by *E. granulosus* infections is between 2 - 4%, by *E. multilocularis* infections between 52 - 94%, after surgical or chemotherapeutical treatment between 10 - 14 %. The serological proof of specific antibodies to Echinococcus spp. with the ELISA allow a diagnosis in 90% of infections. A differentiation between *E. granulosus* and *E. multilocularis* is not possible. Falsenegative results can be obtained when cysts are calcified, false-positive results in case of crossreactions to infections with other parasites (*Fasciola*, *Filariae*). Therefore a combination of ELISA and IFA is recommended.

3. PRINCIPLE OF THE TEST

The IBL-America Echinococcus IgG antibody test kit is based on the principle of the enzyme immunoassay (EIA). Echinococcus antigen is bound on the surface of the microtiter strips. Diluted patient serum or ready-to-use standards are pipetted into the wells of the microtiter plate. A binding between the IgG antibodies of the serum and the immobilized Echinococcus antigen takes place. After a one hour incubation at room temperature, the plate is rinsed with diluted wash solution, in order to remove unbound material. Then ready-to-use anti-human-IgG peroxidase conjugate is added and incubated for 30 minutes. After a further washing step, the substrate (TMB) solution is pipetted and incubated for 20 minutes, inducing the development of a blue dye in the wells. The color development is terminated by the addition of a stop solution, which changes the color from blue to yellow. The resulting dye is measured spectrophotometrically at the wavelength of 450 nm. The concentration of the IgG antibodies is directly proportional to the intensity of the color.

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
8201 Central Ave NE, Suite P, Minneapolis, MN 55432
Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com

4. LIMITATIONS, PRECAUTIONS AND GENERAL COMMENTS

- For *in-vitro* diagnostic use only.
- Do not ingest or swallow! The usual laboratory safety precautions as well as the prohibition of eating, drinking and smoking in the lab have to be followed.
- All sera and plasma or buffers based upon, have been tested respective to HBsAg, HIV and HCV with recognized methods and were found negative. Nevertheless precautions like the use of latex gloves have to be taken.
- Serum and reagent spills have to be wiped off with a disinfecting solution (e.g. sodium hypochlorite, 5%) and have to be disposed of properly.
- All reagents have to be brought to room temperature (18 to 25 °C) before performing the test.
- Before pipetting all reagents should be mixed thoroughly by gentle tilting or swinging. Vigorous shaking with formation of foam should be avoided.
- It is important to pipet with constant intervals, so that all the wells of the microtiter plate have the same conditions.
- When removing reagents out of the bottles, care has to be taken that the stoppers are not contaminated. Further a possible mix-up has to be avoided. The content of the bottles is usually sensitive to oxidation, so that they should be opened only for a short time.
- In order to avoid a carry-over or a cross-contamination, separate disposable pipet tips have to be used.
- No reagents from different kit lots have to be used, they should not be mixed among one another.
- All reagents have to be used within the expiry period.
- In accordance with a Good Laboratory Practice (GLP) or following ISO9001 all laboratory devices employed should be regularly checked regarding the accuracy and precision. This refers amongst others to microliter pipets and washing or reading (ELISA-Reader) instrumentation.
- The contact of certain reagents, above all the stopping solution and the substrate with skin, eye and mucosa has to be avoided, because possible irritations and acid burns could arise, and there exists a danger of intoxication.

5. REAGENTS PROVIDED

Symbol	Components	Volume / Qty.
SORB MT	Echinococcus antigen coated microtiter strips	12
CAL A	Calibrator A (Negative Control)	2 mL
CAL B	Calibrator B (Cut-Off Standard)	2 mL
CAL C	Calibrator C (Weak Positive Control)	2 mL
CAL D	Calibrator D (Positive Control)	2 mL
ENZ CONJ	Enzyme Conjugate	15 mL
SUB TMB	Substrate	15 mL
STOP SOLN	Stop Solution	15 mL
SAM DIL	Sample Diluent	60 mL
WASH SOLN 10x	Washing Buffer (10x)	60 mL

Storage and Stability (refer to the expiry date on the outer box label)

Store kit components at 2-8°C and do not use after the expiry date on the box outer label. Before use, all components should be allowed to warm up to ambient temperature (18-25°C). After use, the plate should be resealed, the bottle caps replaced and tightened and the kit stored at 2-8°C. After the first opening the kit should be used within 3 months, the diluted wash buffer can be kept for 4 weeks at 2-8°C.

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
 8201 Central Ave NE, Suite P, Minneapolis, MN 55432
 Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com

5.1. Microtiter Strips

12 strips with 8 breakable wells each, coated with Echinococcus antigen (Echinococcus granulosus extract from hydatid cysts). Ready-to-use.

5.2. Calibrator A (Negative Control)

2 mL, protein solution diluted with PBS, contains no IgG antibodies against Echinococcus. Addition of 0.01 % methylisothiazolone and 0.01 % bromonitrodioxane. Ready-to-use.

5.3. Calibrator B (Cut-Off Standard)

2 mL human serum diluted with PBS, contains a low concentration of IgG antibodies against Echinococcus. Addition of 0.01 % methylisothiazolone and 0.01 % bromonitrodioxane. Ready-to-use.

5.4. Calibrator C (Weak Positive Control)

2 mL, human serum diluted with PBS, contains a medium concentration of IgG antibodies against Echinococcus. Addition of 0.01 % methylisothiazolone and 0.01 % bromonitrodioxane. Ready-to-use.

5.5. Calibrator D (Positive Control)

2 mL, human serum diluted with PBS, contains a high concentration of IgG antibodies against Echinococcus. Addition of 0.01 % methylisothiazolone and 0.01 % bromonitrodioxane. Ready-to-use.

5.6. Enzyme Conjugate

15 mL, anti-human-IgG-HRP (rabbit), in protein-containing buffer solution. Addition of 0.01 % methylisothiazolone and 0.01 % bromonitrodioxane and 5 mg/L Proclin™. Ready-to-use.

5.7. Substrate

15 mL, TMB (tetramethylbenzidine). Ready-to-use.

5.8. Stop Solution

15 mL, 0.5 M acidic solution. Ready-to-use.

5.9. Sample Diluent

60 mL, PBS/BSA buffer. Addition of 0.095 % sodium azide. Ready-to-use.

5.10. Washing Buffer

60 mL, PBS + Tween 20, 10x concentrate. Final concentration: dilute 1+9 with distilled water. If during the cold storage crystals precipitate, the concentrate should be warmed up at 37°C for 15 minutes.

6. MATERIALS REQUIRED BUT NOT PROVIDED

- 5 µL-, 100 µL- and 500 µL micro- and multichannel pipets
- Microtiter Plate Reader (450 nm)
- Microtiter Plate Washer
- Reagent tubes for the serum dilution
- Deionized water
- Re-usable black lid for covering
- Plastic Bag

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
8201 Central Ave NE, Suite P, Minneapolis, MN 55432
Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com

7. SAMPLE COLLECTION AND HANDLING

Principally serum or plasma (EDTA, heparin) can be used for the determination. Serum is separated from the blood, which is aseptically drawn by venipuncture, after clotting and centrifugation. The serum or plasma samples can be stored refrigerated (2-8°C) for up to 7 days. For a longer storage they should be kept at -20°C. The samples should not be frozen and thawed repeatedly. Lipemic, hemolytic or bacterially contaminated samples can cause false positive or false negative results.

For the performance of the test the samples (not the standards) have to be diluted 1:101 with ready-to-use sample diluent (e.g. 5 µL serum + 500 µL sample diluent).

8. ASSAY PROCEDURE

8.1. Preparation of Reagents

Washing Solution: dilute before use 1+9 with distilled water. If during the cold storage crystals precipitate, the concentrate should be warmed up at 37°C for 15 minutes.

- Strict adherence to the protocol is advised for reliable performance. Any changes or modifications are the responsibility of the user.
- All reagents and samples must be brought to room temperature before use, but should not be left at this temperature longer than necessary.
- Calibrators and unknowns should be assayed in duplicates.
- A standard curve should be established with each assay.
- Return the unused microtiter strips to the plastic bag and store them dry at 2-8°C.

8.2. Assay Steps

1. Prepare a sufficient amount of microtiter wells for the standards, controls and samples as well as for a substrate blank.
2. Pipet 100 µL each of the **diluted** (1:101) unknowns and the **ready-to-use** calibrators and controls respectively into the wells. Leave one well empty for the substrate blank.
3. Cover plate with the re-usable plate cover and incubate at room temperature for 60 minutes.
4. Empty the wells of the plate (dump or aspirate) and add 300 µL of diluted washing solution. This procedure is repeated totally three times. Rests of the washing buffer are afterwards removed by gentle tapping of the microtiter plate on a tissue cloth.
5. Pipet 100 µL each of ready-to-use conjugate into the wells. Leave one well empty for the substrate blank.
6. Cover plate with the re-usable plate cover and incubate at room temperature for 30 minutes.
7. Empty the wells of the plate (dump or aspirate) and add 300 µL of diluted washing solution. This procedure is repeated totally three times. Rests of the washing buffer are afterwards removed by gentle tapping of the microtiter plate on a tissue cloth.
8. Pipet 100 µL each of the ready-to-use substrate into the wells. This time also the substrate blank is pipetted.
9. Cover plate with the re-usable plate cover and incubate at room temperature for 20 minutes in the dark (e.g. drawer).
10. To terminate the substrate reaction, pipet 100 µL each of the ready-to-use stop solution into the wells. Pipet also the substrate blank.
11. After thorough mixing and wiping the bottom of the plate, perform the reading of the absorption at 450 nm (optionally reference wavelength of 620 nm). The color is stable for at least 60 minutes.

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
8201 Central Ave NE, Suite P, Minneapolis, MN 55432
Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com

9. EVALUATION

	OD Value	Corrected OD
Substrate Blank	0.012	
Negative Control	0.048	0.036
Cut-Off Standard	0.408	0.396
Weak Positive Control	1.069	1.057
Positive Control	2.098	2.086

The above table contains only an example, which was achieved under arbitrary temperature and environmental conditions. The described data constitute consequently no reference values which have to be found in other laboratories in the same way.

9.1. Qualitative Evaluation

The calculated absorptions for the patient sera, as mentioned above, are compared with the value for the cut-off standard. If the value of the sample is higher, there is a positive result. For a value below the cut-off standard, there is a negative result. It seems reasonable to define a range of +/-20 % around the value of the cut-off as a grey zone. In such a case the repetition of the test with the same serum or with a new sample of the same patient, taken after 2-4 weeks, is recommended. Both samples should be measured in parallel in the same run.

The positive control must show at least the double absorption compared with the cut-off standard.

9.2. Quantitative Evaluation

The ready-to-use standards and controls of the Echinococcus IgG antibody kit are defined and expressed in arbitrary units (U/mL). This results in an exact and reproducible quantitative evaluation. Consequently for a given patient follow-up controls become possible. The values for controls and standards in units are printed on the QC data sheet.

For a quantitative evaluation the absorptions of the standards and controls are graphically drawn point-to-point against their concentrations. From the resulting reference curve the concentration values for each patient sample can then be extracted in relation to their absorptions. It is also possible to use automatic computer programs. As curve fit point-to-point has to be chosen.

Calibrator B with its concentration of 10 U/mL serves as cut-off standard. Analogous to the qualitative evaluation a range of +/-20% around the cut-off is defined as a grey zone. Thus results between 8 and 12 U/mL are reported as borderline.

10. ASSAY CHARACTERISTICS

Echinococcus ELISA	IgG
Intra-Assay-Precision	9.4 %
Inter-Assay-Precision	10.6 %
Inter-Lot-Precision	7.6 – 11.7 %
Analytical Sensitivity	1.11 U/mL
Recovery	86 - 105 %
Linearity	77 - 124 %
Cross-Reactivity	No cross-reactivity to Mumps, Candida and Yersinia.
Interferences	No interferences to bilirubin up to 0.3 mg/mL, hemoglobin up to 8.0 mg/mL and triglycerides up to 5.0 mg/mL
Clinical Specificity	97 %
Clinical Sensitivity	93 %

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
 8201 Central Ave NE, Suite P, Minneapolis, MN 55432
 Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com

11. REFERENCES

1. Abdel Aal, TM. et al. Studies on the most reactive purified antigen for immuno-diagnosis of hydatid disease. *J. Egypt. Soc. Parasitol.*, 26(2): 297-303 (1996).
2. Aceti, A. et al. The serological diagnosis of human hydatid disease by time-resolved fluoroimmunoassay. *J. Infect.*, 22(2): 135-41 (1991).
3. Auer, H. et al. Combined application of enzyme-linked immunosorbent assay (ELISA) and indirect haemagglutination test (IHA) as a useful tool for the diagnosis and post-operative surveillance of human alveolar and cystic echinococcosis. *Zentralbl. Bakteriol. Mikrobiol. Hyg. [A]*, 270(1-2): 313-25 (1988).
4. Bresson-Hadni, S. et al. Seroepidemiologic screening of Echinococcus multilocularis infection in a European area endemic for alveolar echinococcosis. *Am. J. Trop. Med. Hyg.*, 51(6): 837-46 (1994).
5. El-On, J. et al. Echinococcus granulosus: a seroepidemiological survey in northern Israel using an enzyme-linked immunosorbent assay. *Trans. R. Soc. Trop. Med. Hyg.*, 91(5): 529-32 (1997).
6. Force, L. et al. Evaluation of eight serological tests in the diagnosis of human echinococcosis and follow-up. *Clin. Infect. Dis.*, 15(3): 473-80 (1992).
7. Gottstein, B. et al. Immunodiagnosis of polycystic hydatid disease/polycystic echinococcosis due to Echinococcus vogeli. *Am. J. Trop. Med. Hyg.*, 53(5): 558-63 (1995).
8. Gottstein, B. et al. Sero-epidemiological survey for alveolar echinococcosis (by Em2-ELISA) of blood donors in an endemic area of Switzerland. *Trans. R. Soc. Trop. Med. Hyg.*, 81(6): 960-4 (1987).
9. Ioppolo, S. et al. Immunological responses to antigen B from Echinococcus granulosus cyst fluid in hydatid patients. *Parasite Immunol.*, 18(11): 571-8 (1996).
10. Ito, A. et al. Immunodiagnosis of alveolar echinococcosis by enzyme-linked immunosorbent assay using a partially purified Em18/16 enriched fraction. *Clin. Diagn. Lab. Immunol.*, 4(1): 57-9 (1997).
11. Knobloch, J. et al. Serum antibodies in patients with alveolar echinococcosis before and after therapy. *Trop. Med. Parasitol.*, 36(3): 155-6 (1985).

12. ORDERING INFORMATION

This kit is manufactured for Immuno-Biological Laboratories, Inc. (IBL-America). For ordering information, please contact:

Immuno-Biological Laboratories, Inc. (IBL-America)

8201 Central Ave NE, Suite P

Minneapolis, MN 55432

Toll Free: (888) 523-1246

Fax: (763) 780-2988

www.ibl-america.com

info@ibl-america.com

Manufactured for:






Immuno-Biological Laboratories, Inc. (IBL-America)

8201 Central Ave NE, Suite P, Minneapolis, MN 55432

Toll Free: (888) 523-1246 Fax: (763) 780-2988

www.ibl-america.com / info@ibl-america.com

SYMBOLS USED WITH IBL-AMERICA ASSAYS

Symbol	English	Deutsch	Français	Español	Italiano
	Consult instructions for use	Gebrauchsanweisung beachten	Consulter les instructions d'utilisation	Consulte las instrucciones de uso	Consultare le istruzioni per l'uso
CE	European Conformity	CE-Konformitätskennzeichnung	Conformité aux normes européennes	Conformidad europea	Conformità europea
IVD	In vitro diagnostic device	In-vitro-Diagnostikum	Usage Diagnostic in vitro	Para uso Diagnóstico in vitro	Per uso Diagnostica in vitro
RUO	For research use only	Nur für Forschungszwecke	Seulement dans le cadre de recherches	Sólo para uso en investigación	Solo a scopo di ricerca
REF	Catalogue number	Katalog-Nr.	Numéro de catalogue	Número de catálogo	Numero di Catalogo
LOT	Lot. No. / Batch code	Chargen-Nr.	Numéro de lot	Número de lote	Numero di lotto
	Contains sufficient for <n> tests/	Ausreichend für "n" Ansätze	Contenu suffisant pour "n" tests	Contenido suficiente para <n> ensayos	Contenuto sufficiente per "n" saggi
	Storage Temperature	Lagerungstemperatur	Température de conservation	Temperatura de conservación	Temperatura di conservazione
	Expiration Date	Mindesthaltbarkeitsdatum	Date limite d'utilisation	Fecha de caducidad	Data di scadenza
	Legal Manufacturer	Hersteller	Fabricant	Fabricante	Fabbricante
Distributed by	Distributor	Vertreiber	Distributeur	Distribuidor	Distributore
Content	Content	Inhalt	Conditionnement	Contenido	Contenuto
Volume/No.	Volume / No.	Volumen/Anzahl	Volume/Quantité	Volumen/Número	Volume/Quantità

Manufactured for:

Immuno-Biological Laboratories, Inc. (IBL-America)
 8201 Central Ave NE, Suite P, Minneapolis, MN 55432
 Toll Free: (888) 523-1246 Fax: (763) 780-2988
www.ibl-america.com / info@ibl-america.com